

Oral Temperatures with SureTemp® Plus

Models 692/690




1. Ensure that the probe with the blue ejection button and blue probe well are installed.
2. Remove the probe from the probe well
3. Observe the Oral Mode indicator on the display (flashing head icon). ↑

If this icon is not flashing, press the Mode  Selection button until the head icon appears.

4. Load a Welch Allyn probe cover.

5. Place the probe tip deep into the patient's sublingual pocket as shown.

6. Do not hand the probe to the patient to place in his or her own mouth.
7. Always hold the probe in place, maintaining tissue contact until temperature is complete.

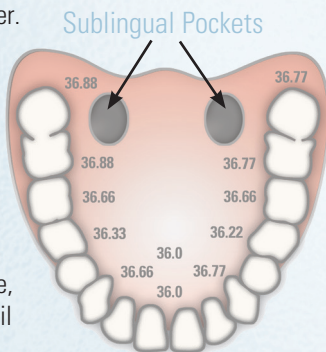

WelchAllyn®

Advancing Frontline Care™

Tips for Taking Oral Temperatures with SureTemp®

1. Always allow patient to ask questions prior to placing probe in mouth. Encourage them not to speak while thermometer is taking a temperature.
2. Please inform your patient that biting the probe tip while taking the temperature may result in damage to the probe.
3. Use only Welch Allyn disposable probe covers. Use of any other probe cover may result in inaccurate temperature readings.
4. If you question a reading, recheck the temperature using the monitor mode on your thermometer. Take an oral temperature as described.
 - > Continue to hold the probe in place after the temperature displays
 - > Press the Mode Selection  until the Monitor Mode  indicator appears on the screen
 - > Continue taking the temperature for approximately three minutes
 - > Long-term continuous monitoring is not recommended in any mode.

The following may interfere with accurate readings for up to 20 minutes:

- | | |
|------------------|--|
| > Brushing teeth | > Ingesting hot or cold liquids |
| > Eating | > Chewing gum or mints |
| > Smoking | > Activity such as climbing stairs or running. |

Welch Allyn UK Ltd.
Clinitron House
Excelsior Road
Ashby-de-la-Zouch LE65 1JG
Tel: 0207 365 6780

WelchAllyn®

Advancing Frontline Care™