

CP 150 12-lead resting electrocardiograph



Directions for use

Software version 2.00.XX

© 2013 Welch Allyn, Inc. To support the intended use of the product described in this publication, the purchaser of the product is permitted to copy this publication, for internal distribution only, from the media provided by Welch Allyn.

Caution: Federal US law restricts sale of the device identified in this manual to, or on the order of, a licensed physician.

Welch Allyn assumes no responsibility for any injury, or for any illegal or improper use of the product, that may result from failure to use this product in accordance with the instructions, cautions, warnings, or indications for use published in this manual.

Welch Allyn is a registered trademark of Welch Allyn, Inc. CP 150, and CardioPerfect are trademarks of Welch Allyn, Inc.

Patent information

For patent information, please visit www.welchallyn.com/patents.

Software in this product is copyright Welch Allyn or its vendors. All rights are reserved. The software is protected by United States of America copyright laws and international treaty provisions applicable worldwide. Under such laws, the licensee is entitled to use the copy of the software incorporated within this instrument as intended in the operation of the product in which it is embedded. The software may not be copied, decompiled, reverse-engineered, disassembled or otherwise reduced to human-perceivable form. This is not a sale of the software or any copy of the software; all right, title and ownership of the software remains with Welch Allyn or its vendors.

Welch Allyn Technical Support:


<http://www.welchallyn.com/about/company/locations.htm>

105755 (CD)
DIR 80018007 Ver. B

Material Number 720157, DIR 80018007 Ver. B



Welch Allyn
4341 State Street Road
Skaneateles Falls, NY 13153-0220 U.S.A
www.welchallyn.com

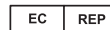

Regulatory Affairs Representative
Welch Allyn Limited
Navan Business Park
Dublin Road
Navan, County Meath
Republic of Ireland



WelchAllyn[®]
Advancing Frontline Care[™]

Contents

Introduction	1
About this document	1
Intended use	1
Indications for use	1
Contraindications	1
Description	2
Features	2
Controls, indicators, and connectors	3
Symbols	5
General warnings	7
General cautions	9
Setup	11
Connecting the patient cable	11
Loading the thermal paper	11
Powering the electrocardiograph	12
Verifying proper operation	12
Connecting to a CardioPerfect workstation	13
To connect to a network-connected workstation via Ethernet cable	13
ECG home screen	15
ECG home screen	15
ECG tests	19
Attaching the leads to the patient	19
Electrode locations	21
Viewing electrode placement	22
Performing an Auto ECG test	22
Assigning an Auto ECG test to the worklist	27
Assign an Auto ECG test to the worklist	27
Saved tests	29
Searching for saved tests	29
Managing the saved tests	29
Saving a test as a PDF	30
Managing the worklist	30
Settings	31
Viewing or changing the ECG settings	31
Selecting the device information	33

Maintenance	35
Cleaning the equipment	35
Inspecting the equipment	36
Testing the electrocardiograph	36
Replacing the battery	37
Replacing the AC fuses	37
Storing the equipment	38
Discarding the equipment	38
Troubleshooting	39
Lead-quality problems	39
System problems	42
Limited warranty	44
Service policy	45
EMC guidance and manufacturer's declarations	47
EMC compliance	47
Emissions and immunity information	47
Specifications	49

Introduction

About this document

This document is written for clinical professionals with a working knowledge of medical procedures and terminology required for monitoring cardiac patients.

Before using the electrocardiograph for clinical applications — or before setting up, configuring, troubleshooting, or servicing the electrocardiograph — you must read and understand this document and all other information that accompanies the electrocardiograph and related options or accessories.

Intended use

The Welch Allyn electrocardiography products (subject devices) are intended for use by trained operators in health facilities. The subject devices provide the following diagnostic functions:

- Acquiring, viewing, storing, and printing ECG waveforms using ECG front-end modules (patient cables) and associated accessories that provide signal acquisition for up to twelve (12) leads of patient ECG waveforms through surface electrodes adhered to the body.
- Using optional algorithms to generate measurements, data presentations, graphical presentations, and interpretative statements on an advisory basis. These are presented for review and interpretation by the clinician based upon knowledge of the patient, the result of physical examination, the ECG tracings, and other clinical findings.

Indications for use

The electrocardiograph is one of the tools that clinicians use to evaluate, diagnose, and monitor patient cardiac function.

The 12-lead ECG interpretive algorithm provides a computer-generated analysis of potential patient cardiac abnormalities which must be confirmed by a physician with other relevant clinical information.

Contraindications

The electrocardiograph has no known contraindications.

Description

- The electrocardiograph allows users to perform 12-lead ECG measurements and analysis.
- The electrocardiograph supports STAT, Auto, and Rhythm test types.
- The electrocardiograph provides the ability to print test records on an internal printer.
- The electrocardiograph allows storage of test records in device memory, external storage media, and external software applications.
- The electrocardiograph allows users to enter patient demographic data into the electrocardiograph memory to be recalled for a test later that day.

Features

Pacemaker detection

The software detects the possible presence of a pacemaker. If you confirm that the patient has a pacemaker, the ECG report includes no interpretation, and it indicates that a pacemaker was detected.


Automatic ECG interpretation (optional)

The optional MEANS interpretation algorithm, developed by the University of Rotterdam in the Netherlands, provides automatic analysis of ECG tests. For more information, see the MEANS Physicians' Manual or the PEDMEANS Physicians' Manual on the CD that came with your electrocardiograph. The MEANS algorithm is used for adult patients 18 years and older. The PEDMEANS algorithm is used for pediatric patients from 1 day through 17 years old.




CAUTION Check for the presence of a pacemaker before using ECG with interpretation.

Controls, indicators, and connectors

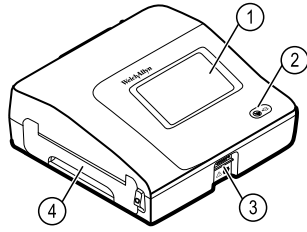

No.	Feature	Description
1	LCD screen	800 x 480 pixels color touchscreen provides a graphical user interface.
2	Power switch and LED	Power-on/Standby switch. The LED indicates the charging status when connected to AC power: <ul style="list-style-type: none"> • Green: The battery is charged. • Amber: The battery is charging.
3	Patient cable connector	Provides connection for patient cable.
4	Printer	Printer provides a printout of patient Auto ECG, Stat ECG, or Rhythm ECG.

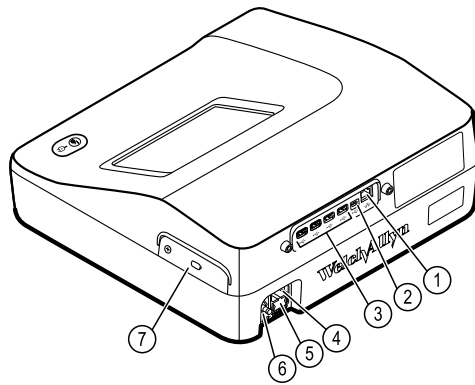





Back view
















No.	Feature	Description
1	Ethernet connector	Provides a hardwired connection to the computer network. The LEDs indicate active network status when the ethernet cable is connected to a network.
2	Clients USB	USB, type "mini B." Provides connection to an enabled host.
3	Host USB	USB, type "A." Provides four host USB connections for optional accessories.
4	Power connection	Provides an external AC power connection.
5	AC fuse	Provides access to AC fuse.
6	Ground lug (equipotential terminal)	Provided for electrical safety testing and as a means for connection of a potential equalization conductor.
7	Battery compartment (behind cover)	Houses the Li-ion battery.

Symbols

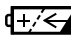

Documentation symbols

	<p>WARNING The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death.</p>		<p>Caution The caution statements in this manual identify conditions or practices that could result in damage to the equipment or other property, or loss of data. This definition applies to both yellow and black and white symbols.</p>
	<p>Consult Directions for Use, Electronic version available at Welchallyn.com, or Hard copy DFU available from Welch Allyn within 7 days.</p>		<p>Meets essential requirements of European Medical Device Directive 93/42/EEC.</p>











Power symbols

	<p>Power on/standby</p>	
	<p>(LED off) No AC power is present</p>	 <p>The battery is absent or faulty</p>
	<p>(LED green) AC power is present, and battery is charged. (LED amber) AC power is present, and battery is charging.</p>	 <p>The battery is charged to the indicated level</p>
	<p>Alternating current</p>	 <p>Battery is charging</p>
	<p>Dangerous voltage</p>	 <p>Battery door</p>
	<p>Fuse</p>	 <p>Rechargeable battery</p>
	<p>Protective earth</p>	 <p>Nonionizing electromagnetic radiation</p>
	<p>Equipotential terminal</p>	 <p>AC input power</p>



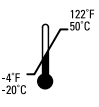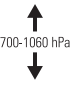



Connectivity symbols

	USB		Ethernet (RJ-45)
---	-----	---	-------------------

Shipping, storing, and environment symbols

	This end up		Keep dry
	Fragile		Relative humidity limit
	Temperature limits		Atmospheric air-pressure limits
	Separate the battery from other disposables for recycling		Recycle
	Separate the device from other disposables for recycling. See www.welchallyn.com/weee .		China RoHS (restriction of hazardous substances) symbols for control of pollution caused by electronic information products. 5-year environment-friendly use period (EFUP) for batteries. 10-year EFUP for the device. For details, see the accompanying documentation.
Li-ion	Lithium ion battery		

Miscellaneous symbols

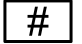	Manufacturer		Defibrillation-proof Type CF applied parts
	Reference/Model number		Serial number
	Reorder/Catalog number		Batch code



Intertek ETL listed



Do not reuse

General warnings

Warnings indicate conditions or practices that could lead to illness, injury, or death.

Warnings related to the environment



WARNING The power cord is considered the disconnect device for isolating this equipment from supply mains. Do not position the equipment so that it is difficult to reach or disconnect.



WARNING To avoid a possible explosion, do not use the electrocardiograph in the presence of flammable anesthetics: mixtures containing air, oxygen, or nitrous oxide.



WARNING When transporting the electrocardiograph on a cart, tuck the patient cable away to keep them clear of the wheels and to minimize trip hazards.

Warnings related to accessories and other equipment



WARNING To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.



WARNING For operator and patient safety, peripheral equipment and accessories that can come in direct patient contact must be in compliance with all appropriate safety, EMC, and regulatory requirements.



WARNING All signal input and output (I/O) connectors are intended for connection of only devices complying with IEC 60601-1, or other IEC standards (for example, IEC 60950), as appropriate to the device. Connecting additional devices to the electrocardiograph might increase chassis or patient leakage currents.



WARNING The electrocardiograph has not been designed for use with high-frequency (HF) surgical equipment and does not protect against hazards to the patient.



WARNING Defective batteries can damage the electrocardiograph. Visually inspect the battery at least monthly, if the battery shows any signs of damage or cracking, it must be replaced immediately and only with a battery approved by Welch Allyn.



WARNING Improper disposal of batteries may create an explosion or contamination hazard. Never dispose of batteries in refuse containers. Always recycle batteries according to local regulations.



WARNING All signal input and output (SIP/SOP) connectors should not be contacted by patient directly and indirectly through the user during operation.

Warnings related to using the electrocardiograph



WARNING No modification of this equipment is allowed.



WARNING This device captures and presents data reflecting a patient's physiological condition. When reviewed by a trained physician or clinician, this data can be useful in determining a diagnosis. However, the data should not be used as a sole means for determining a patient's diagnosis or prescribing treatment.



WARNING To provide CF protection use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.



WARNING To avoid serious injury or death, take these precautions during patient defibrillation:

- Avoid contact with the electrocardiograph, patient cable, and patient.
- Verify that the patient leads are properly connected.
- Place defibrillator paddles properly in relation to electrodes.
- After defibrillation, pull each patient lead out of the patient cable and inspect the tips for charring (black carbon marks). If there is any charring, the patient cable and individual leads must be replaced. If there is no charring, fully reinsert the leads into the patient cable. (Charring can occur only if a lead is not fully inserted into the patient cable before defibrillation.)



WARNING To prevent the spread of infection, take these precautions:

- Dispose of single-use components (for example, electrodes) after using them once.
- Regularly clean all components that come in contact with patients.
- Avoid ECG testing for patients with open, infectious sores.



WARNING Avoid positioning any leads or cables so that they could easily trip someone or become wrapped around a patient's neck.



WARNING To ensure safe use of the device, follow the documented maintenance procedures.



WARNING Only qualified service personnel should attempt to repair the electrocardiograph. In case of a malfunction, call Technical Support.



WARNING Do not perform ST segment analysis on the ECG screen display, since these ECG representations are scaled. Make manual measurements of ECG intervals and magnitudes on printed ECG reports only.



WARNING To maintain diagnostic accuracy and to comply with IEC 60601-02-51 and IEC 60601-02-25, do not scale (re size) when sending a saved ECG to an external printer.



WARNING To avoid injury, do not touch the print head immediately after printing. It might be hot.

General cautions

Cautions indicate conditions or practices that could damage the equipment or other property.



CAUTION Federal US law restricts sale of the device identified in this manual to, or on the order of, a licensed physician.



CAUTION When removing the electrocardiograph from storage, allow it to thermally stabilize to surrounding environmental conditions before using it.



CAUTION To prevent possible damage, do not use sharp or hard objects to press the touch screen or the buttons. Only use fingertips.



CAUTION Do not expose the patient cable to strong ultraviolet radiation.



CAUTION Do not pull or stretch the patient cable. Doing so could result in mechanical or electrical failures. Form the patient cable into a loose loop before storing.



CAUTION Avoid positioning the patient cable where it might get pinched, stretched, or stepped on. Otherwise, measurements might no longer be accurate, and repair might be necessary.



CAUTION Using the equipotential terminal for anything but grounding purposes may contribute to damage of the device.



CAUTION Use only parts and accessories, including thermal paper, that were supplied with the device and available through Welch Allyn. The use of accessories other than those specified may result in degraded performance or unsafe use of this device.



CAUTION Portable and mobile RF communications equipment can affect the performance of the electrocardiograph.



CAUTION The electrocardiograph meets the Class A requirements of IEC 60601-1-2 regarding incidental emission of radio frequency interference. As such it is suitable for use in commercial grade electrical environments. If the electrocardiograph is used in residential grade electrical environments and you experience incidental interference with other equipment that uses radio frequency signals to operate, minimize the interference.



CAUTION Other medical equipment—including but not limited to defibrillators, ultrasound machines, pacemakers, and other stimulators—may be used simultaneously with the electrocardiograph. However, such devices may disturb the electrocardiograph signal.



CAUTION The power cord must be disconnected from AC power before cleaning, maintaining, transporting, or servicing.




CAUTION The requirements of AAMI EC11, Section 3.2.7.2, Frequency and Impulse Response, for an impulse triangle waveform may be impacted by up to 5 milliseconds of small amplitude dampened ringing immediately after the impulse when the muscle filter (35 Hz) is turned on or a small amplitude offset when the baseline filter (0.5 Hz) is turned on. These filters, in any combination turned on or off, meet the AAMI requirements. Measurements performed by the optional interpretation algorithm are unaffected by any filter selections.

Note

The entire patient cable, up to and including the electrodes are considered to be an Applied Part.

Setup

Connecting the patient cable

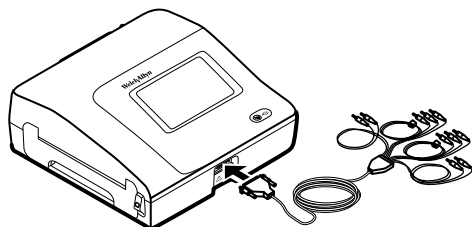

WARNING Do not allow the conductive parts of the patient cable, electrodes, or associated connections of defibrillation-proof applied parts, including the neutral conductor of the patient cable and electrodes, to come into contact with other conductive parts, including earth ground. Otherwise, an electrical short might result, risking electric shock to patients and damage to the device.



WARNING To avoid injury to the patient or damage to the device, never plug patient leads into any other device or wall outlet.



WARNING To provide CF protection use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.




CAUTION Always connect the patient cable and the leads properly during defibrillation. Otherwise, the connected leads could be damaged.

Loading the thermal paper

The electrocardiograph prints on z-fold thermal paper.

- Store the paper in a cool, dry, dark place.
- Do not expose it to bright light or UV sources.
- Do not expose it to solvents, adhesives, or cleaning fluids.
- Do not store it with vinyls, plastics, or shrink wraps.

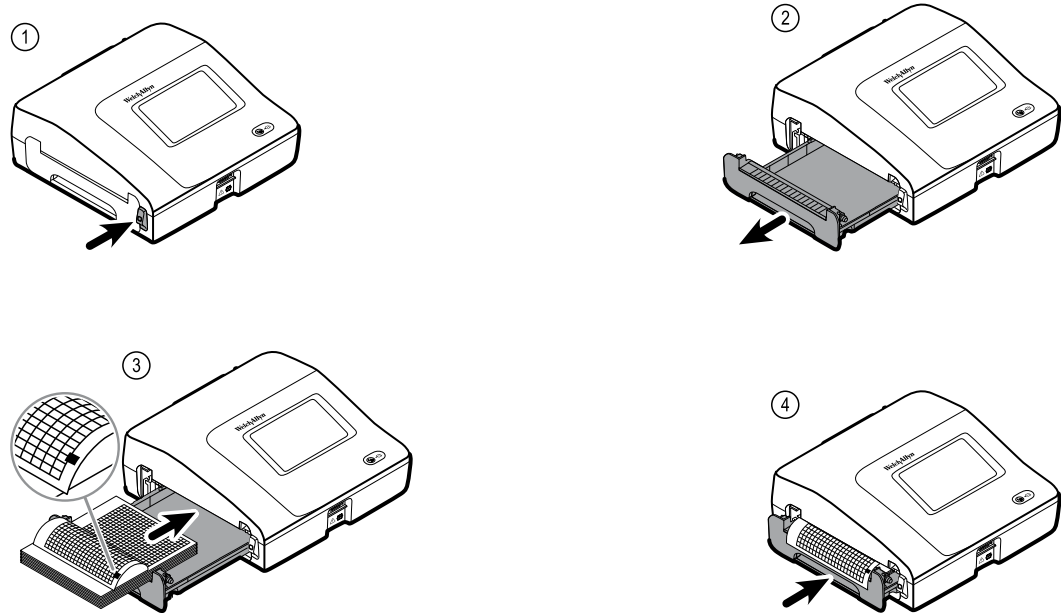

Powering the electrocardiograph

The electrocardiograph runs on AC or battery power. Plug the electrocardiograph into AC power as often as possible so that the built-in charger can keep the battery charged. Regardless of the battery condition, you can use the electrocardiograph whenever it is plugged in.



WARNING When you use AC power, always plug the electrocardiograph into a hospital-grade outlet to avoid the risk of shock.




WARNING If the integrity of the building's safety ground is in doubt, operate this device on battery power to avoid the risk of shock.

To turn on or turn off



Press

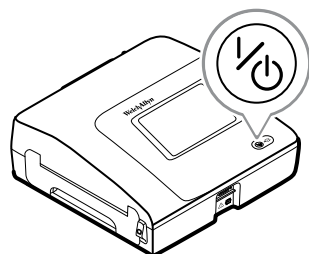

Verifying proper operation

To ensure the accuracy of test data, you should verify proper operation of the electrocardiograph before it is used for the first time with patients. You should reverify proper operation once a year.

To verify proper operation

1. Use an ECG simulator to acquire and print a standard 12-lead ECG of known amplitude and rate.
2. Look for the following indications of proper operation:
 - Printing should be dark and even across the page.
 - There should be no evidence of print-head dot failure (no printing breaks forming horizontal streaks).
 - Thermal paper should move smoothly and consistently during printing.
 - The waveforms should have no distortion or excessive noise.
 - The amplitude and rate of the waveforms should match the input value from the ECG simulator.
 - Z-fold thermal paper should stop with perforations near the tear bar, indicating proper cue-sensor operation.
3. If you see any indications of improper operation, contact Welch Allyn Technical Support.

Connecting to a CardioPerfect workstation


You can connect an electrocardiograph to a Welch Allyn CardioPerfect workstation via USB cable or via Ethernet cable. Software provided in the Connex Software Connectivity Kit is required to communicate with the workstation. After you connect, you can perform several functions from the electrocardiograph, including:

- Send ECG tests to the workstation.
- Search for patient data in the workstation database.

Cables longer than 3 meters have not been verified for use with the electrocardiograph. Do not use cables longer than 3 meters.

To connect to a network-connected workstation via Ethernet cable

1. Connect the electrocardiograph to the same network as the workstation. If you need help, consult your network administrator.

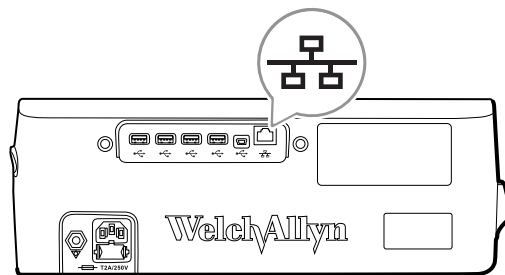

WARNING To reduce the risk of fire and shock hazards, connect Ethernet cables within the boundaries of a single building. Ethernet cables that span multiple buildings can introduce fire or shock hazards unless fiber optic cables or lightning arrestors are properly installed or other appropriate safety measures are taken.


2. Go to the Network screen
 - a. Touch the **Settings** tab.
 - b. Touch the **Device** tab.
 - c. Touch the **Network** button.
3. Fill in the screen.
 - If DHCP is enabled for the network, select the **DHCP** checkbox.
 - If DHCP is not enabled, fill in the fields: Network IP address, Subnet mask, and Default gateway.
4. Touch the **Test with ping** button.
5. Connect to the network.
 - a. In the **Host Address** field, enter the host IP address.
 - b. Touch the **Ping Test** button. The electrocardiograph pings the host.
6. Choose one of these to specify how to get DCP server IP information:
 - **Obtain server IP information automatically:** Select this option if you want the device to get the IP address, or if you don't know the server's IP address.
 - **DCP IP address:** Fill in this field if your DCP server IP address is available.
 - Fill in the **UDP broadcast port** field.
7. Touch **Test connection**.
8. Fill in the Remote file server address, User ID, and password.
9. Touch **Test remote folder**.

The electrocardiograph connects to the remote file server. The electrocardiograph is ready to send tests to the remote file server. (To set up the workstation, see the instructions that came with the workstation connectivity software.)

ECG home screen

ECG home screen

The ECG home screen includes the following areas:

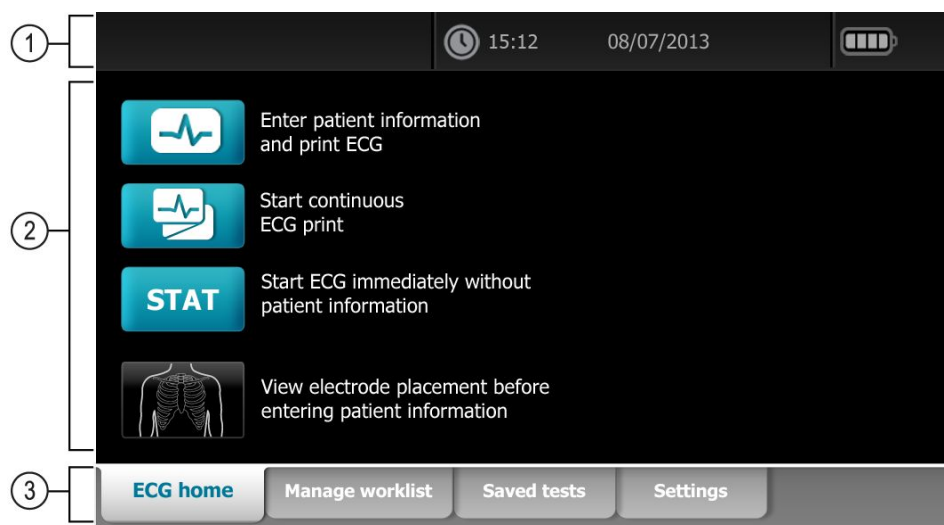

Item	Area
1	Device status
2	Content
3	Navigation

Device status area

The Device status area, located at the top of the ECG home screen, displays:

- Time and date
- Battery status
- Error or information messages. These items are displayed until the condition has been resolved.

Content area

The Content area includes 3 test selection buttons and a preview selection button:

- **Auto ECG**
- **Rhythm ECG**
- **Stat ECG**
- **Electrode Placement**

The content area also provides shortcuts to several controls.

About the test types

Auto ECG



A report typically showing a 10-second acquisition of 12 leads of ECG information combined with patient data, measurements, and optional interpretation. Auto ECGs can be saved to the electrocardiograph's test directory or to a USB mass-storage device.

Rhythm ECG



A continuous, real-time printout of rhythm strips with a user-defined lead configuration. Rhythm ECGs are printouts only. They cannot be saved.

Stat ECG

An auto ECG that starts without waiting for you to enter patient data.



Navigation area


The Navigation area includes the following tabs:

- **ECG home:** Displays ECG test types and provides shortcuts to several controls.
- **Manage worklist :** Includes patient data entered manually or orders downloaded when connected to a hospital information system.
- **Saved tests:** Accesses the patient ECG tests.
- **Settings:** Accesses device configuration settings.

To navigate to a tab, touch the tab in the Navigation area with the corresponding name. The active tab is highlighted.

Example ECG report

ID: STAT_e131fddf Gender: Page: 1
Name: Race: Unknown 03/10/2012 08:09:54AM warning: age not available, assumed 35 years
Age: PR: 92/168 ms warning: sex not available, assumed male
Weight: QRS: 92 ms sinus rhythm
Height: QT/QTc: 396/396 ms probable septal infarct
 P/QRS/T axis: 28/53/40 deg Abnormal ECG
 Heart rate: 60 bpm Unconfirmed Report

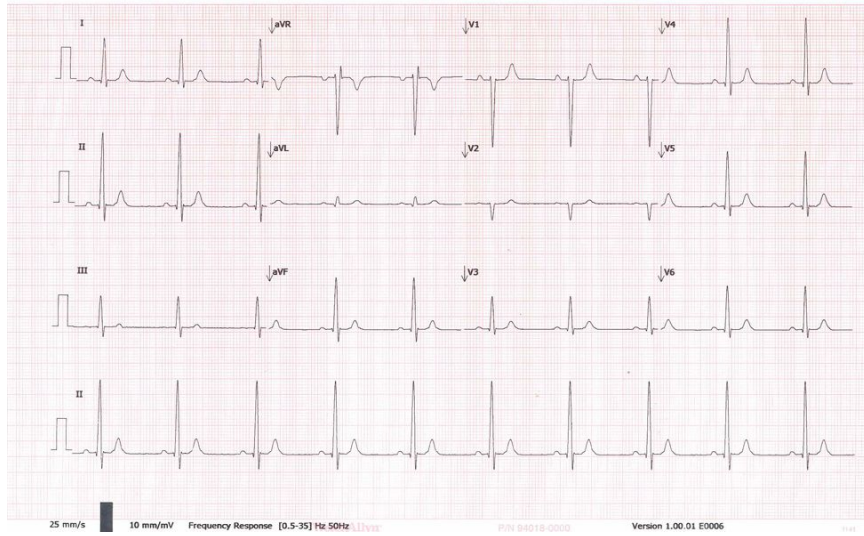

ECG tests

Attaching the leads to the patient


Proper lead attachment is important for a successful ECG. The most common ECG problems are caused by poor electrode contact and loose leads. Follow your local procedures for attaching the leads to the patient. Here are some common guidelines.




WARNING Electrodes can cause allergic reactions. To avoid this, follow the electrode manufacturer's directions.

To attach the leads to the patient

1. Prepare the patient.
 - Describe the procedure. Explain the importance of holding still during the test. (Movement can create artifact.)
 - Verify that the patient is comfortable, warm, and relaxed. (Shivering can create artifact.)
 - Put the patient in a reclining position with the head slightly higher than the heart and legs (the semi-Fowler's position).

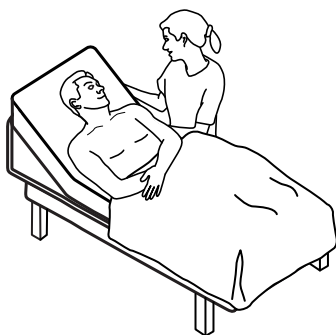


2. Select the electrode locations. (See the "Electrode locations" chart.)
 - Look for flat areas.
 - Avoid fatty areas, bony areas, and major muscles.
3. Prepare the electrode locations.
 - Shave or clip the hair.
 - Thoroughly clean the skin, and lightly rub it dry. You may use soap and water, isopropyl alcohol, or skin preparation pads.
4. Attach the lead wires to the electrodes.
5. Apply the electrodes to the patient.

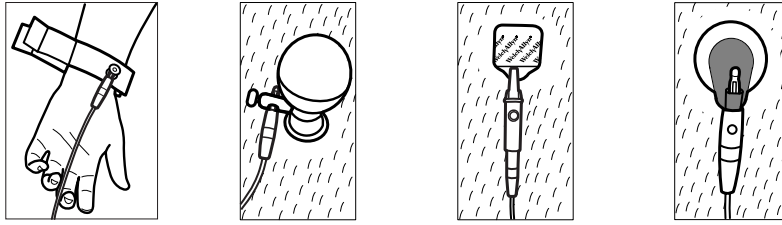

Electrode examples, left to right: arm clamp (reusable), Welsh cup (reusable), tab electrode (disposable), monitoring electrode (disposable).

- **For reusable electrodes:** Use electrode paste, gel, or cream to cover an area the size of each electrode but no larger. Secure the arm and leg clamps. Apply the Welsh cups (suction electrodes) to the chest.
- **For disposable tab electrodes:** Place the electrode tab between the “jaws” of the connector. Keep the tab flat. Verify that the metal portion of the connector makes contact with the skin side of the electrode tab.
- **For all disposable electrodes:** Lightly tug on the connector to ensure that the lead is securely attached. If the electrode comes off, replace it with a new electrode. If the connector comes off, reconnect it.

Electrode locations

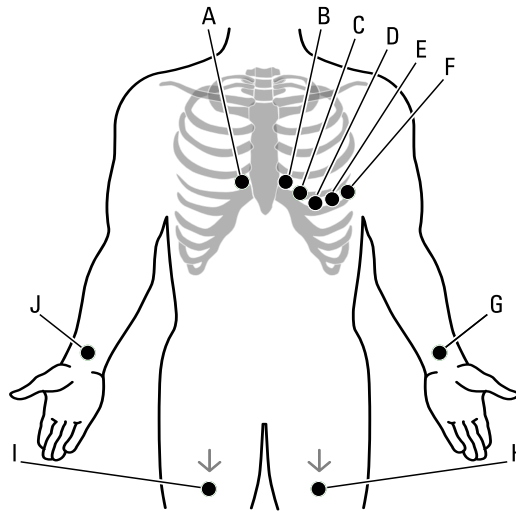




	AHA	IEC	Location
A	V1 (red)	C1 (red)	Fourth intercostal space at the right sternal border.
B	V2 (yellow)	C2 (yellow)	Fourth intercostal space at the left sternal border.
C	V3 (green)	C3 (green)	Midway between V2 and V4.
D	V4 (blue)	C4 (brown)	Fifth intercostal space to the left of the midclavicular line.
E	V5 (orange)	C5 (black)	Anterior axillary line at the same horizontal level as V4.
F	V6 (purple)	C6 (purple)	Mid-axillary line at the same horizontal level as V4 and V5.
G	LA (black)	L (yellow)	Just above the left wrist on the inside of the arm.
H	LL (red)	F (green)	Just above the left ankle.
I	RL (green)	N (black)	Just above the right ankle.
J	RA (white)	R (red)	Just above the right wrist on the inside of the arm.

Viewing electrode placement

1. Touch the  (ECG preview) button.

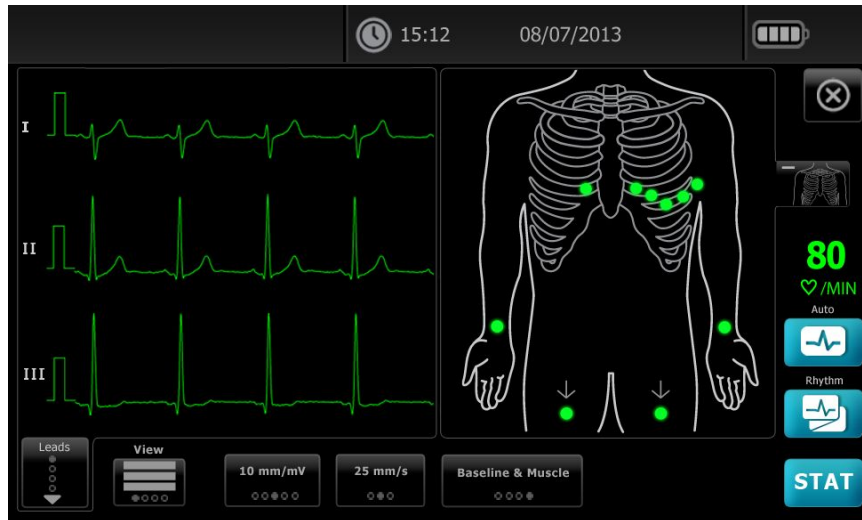The ECG preview screen appears.




2. Touch the  (torso) button to enlarge the lead placement image or touch  to close it.

Performing an Auto ECG test





CAUTION Patient data is not saved until the ECG test is completed.

Note The ECG configuration settings can be changed in the Settings tab. The following settings may appear differently if the default settings have been modified.


1. Touch the  (Auto ECG) button. The Summary tab appears.
2. Enter the following patient information as desired:
 - Patient ID. Touch the **OK** button.
 - Birth date. Touch the **OK** button.
 - Last name. Touch the **OK** button.
 - First name. Touch the **OK** button.
 - Middle Initial. Touch the **OK** button.


Note If the patient has a pacemaker touch the *Pacemaker present* button.

3. Touch the  (Next) button.
4. Enter the following patient information as desired:
 - Gender
 - Race

- Height. Touch the **OK** button.
 - Weight. Touch the **OK** button.
 - Comments. Touch the **OK** button.
5. Touch the  (Next) button.
 6. Attach the leads to the patient.

Note

If desired, touch the  (torso) button to enlarge the lead-placement screen. Any flashing dots on the ECG preview screen indicate unattached or poorly attached leads.

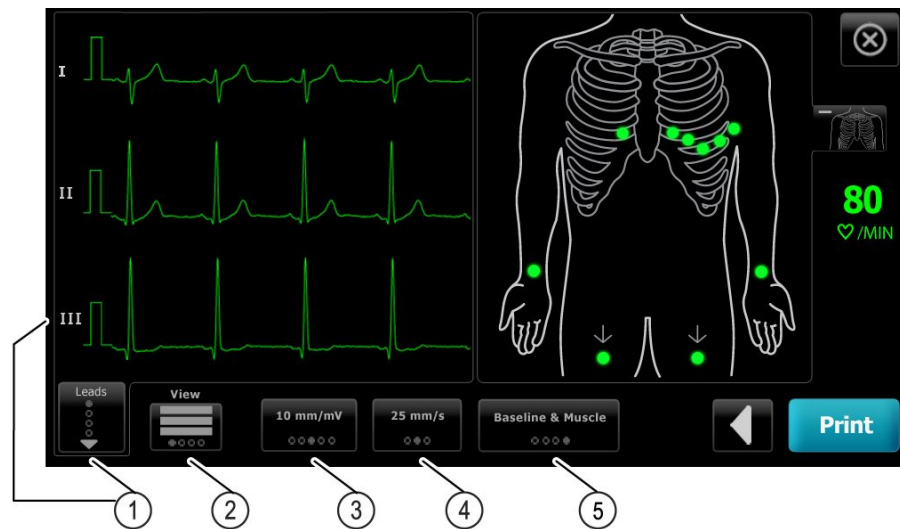

Item	Button
1	Leads button
2	ECG preview button
3	Gain button (size)
4	Speed button
5	Filters button

7. If an *Artifact* message appears, minimize the artifact, as described under Troubleshooting. You might need to ensure that the patient is comfortably warm, reprepare the patient’s skin, use fresh electrodes, or minimize patient motion.
8. (Optional) Adjust the waveforms, using the buttons to cycle through the following options:
 - leads displayed
 - ECG preview format
 - gain (size)
 - speed
 - filters
9. Touch the **Print** button to perform the Auto ECG test.

If *Print preview on* is selected in the ECG configuration settings, a single screen of the ECG Report Print Preview appears. Touch **Print** to continue with the Auto ECG test or touch **Retest** to discard the test and return to the previous screen.

10. If a *Waiting for 10 seconds of quality data* message appears, at least 10 seconds of ECG data have been collected with excessive artifact. Time requirements in the message may vary, based upon selected print format. Minimize the artifact, as described under Troubleshooting. Then wait for the test to print. If necessary, you can override the wait time and print the available data immediately, but the result might be an incomplete or poor-quality test.
11. After the test prints, select the desired option: **Reprint**, **Save**, **Retest**, or **Rhythm**. If the *Auto Save* setting is turned off, touch **Save** to save the test. Select one of the following locations:
 - Local (internal memory)
 - USB mass storage device (Any tests that you save to a USB mass storage device can be retrieved only from a CardioPerfect workstation.)
 - Workstation
 - Remote file location
12. Touch **Reprint** to print the test again, touch **Retest** to discard the test and start over, or touch **Exit**.




WARNING To avoid the risk of associating reports with the wrong patients, make sure that each test identifies the patient. If any report does not identify the patient, write the patient identification information on the report immediately following the ECG test.

Performing an Auto ECG test using the Search tab





CAUTION Patient data is not saved until the ECG test is completed.

Note The ECG configuration settings can be changed in the Settings tab. The following settings may appear differently if the default settings have been modified.

1. Touch the  (Auto ECG) button. The Summary tab appears.
2. Search for patient.

The Search tab gives you access to patient data in the Saved tests directory or in a connected database (CardioPerfect workstation or EMR).

- Touch the **Search** tab.
- Enter the Patient ID or Last name.
- Touch the **OK** button.
- Touch the **Search** button.
- Touch the patient's name or ID in the Patient name/Patient ID field.
- Touch the **Select** button.
- Touch the  (Next) button to review or edit patient information.
- Touch the  (Next) button again.




3. Attach the leads to the patient.
4. Touch the **Print** button to perform the Auto ECG test.

Performing an Auto ECG test using the Worklist tab






CAUTION Patient data is not saved until the ECG test is completed.

Note The ECG configuration settings can be changed in the Settings tab. The following settings may appear differently if the default settings have been modified.

1. Touch the  (Auto ECG) button. The Summary tab appears.
2. Touch the **Worklist** tab.
The worklist is downloaded from the EMR.
3. Select the patient from the Worklist.
 - Touch the Order ID, Patient ID, Patient name, or Birth date in the Patient ID/ Patient name field.
 - Touch the **Select** button.
 - Touch the  (Next) button to review or edit patient information.
 - Touch the  (Next) button again.
4. Attach the leads to the patient.
5. Touch the **Print** button to perform the Auto ECG test.
6. After the test prints, select the desired option: **Reprint**, **Save**, **Retest**, or **Rhythm**.

Performing a Rhythm ECG test after an Auto ECG test

1. Touch the  (Auto ECG) button.
2. Enter the patient information.
 - Touch the  (Next) button to review or edit patient information.
 - Touch the  (Next) button again.
3. Attach the leads to the patient.
4. Touch the **Print** button to perform the Auto ECG test.
5. After the test prints, touch the **Rhythm** button.
If you are prompted to save the Auto ECG test, select Local, USB mass storage device, Workstation, or Remote file location. Touch **Save**.
6. Touch **Start** to begin the Rhythm ECG test.
Touch **Stop** once the desired length of real-time rhythm strips have printed.

Assigning an Auto ECG test to the worklist

You can assign an Auto ECG test to the worklist if patient demographic fields are left blank.




Assign an Auto ECG test to the worklist



CAUTION Patient data is not saved until the ECG test is completed.

Note If an Auto ECG test is performed without entering any patient demographics, this test can be assigned to a patient in the worklist after the test is printed.

Note To use the assign feature the *Test assignment on* setting has to be turned on.

1. Touch the  (Auto ECG) button. The Summary tab appears.
2. Touch the  (Next) button.
3. Touch the  (Next) button.
4. Attach the leads to the patient.
5. Touch the **Print** button to perform the Auto ECG test.
6. After the test prints, touch the **Assign** button.
7. Touch Order ID, Patient ID, Patient name, or Birth date.
8. Touch the **Select** button.

If you are prompted to save the Auto ECG test, select Local, USB mass storage device, Workstation, or Remote file location. Touch **Save**.



WARNING To avoid the risk of associating reports with the wrong patients, make sure that each test identifies the patient. If any report does not identify the patient, write the patient identification information on the report immediately following the ECG test.

Saved tests

Searching for saved tests

Search for Saved Tests by:

- Date
- Last name
- Patient ID
- Test type
 - All
 - Unconfirmed
 - Unprinted
 - Unsent

Once retrieved, Saved Tests can be printed, deleted, edited, or sent to a USB storage device or a Workstation.

Managing the saved tests

Saved tests are a group of ECG tests that have been saved in the electrocardiograph memory.

With all electrocardiograph models, you can delete or print saved tests. You can also do the following:

- Edit the patient data in Saved Tests.
- Send saved tests to a USB mass-storage device, a remote file location, or to a Welch Allyn CardioPerfect workstation. (Any tests that you send to a USB mass-storage device can be retrieved only from a CardioPerfect workstation.)

To manage saved tests

1. Touch the **Saved tests** tab.
2. Enter data into the **Date from**, **Last name**, or **Patient ID** field, or select the check box to search the **Test Type** for **All**, **Unconfirmed**, **Unprinted**, or **Unsent** tests.
3. Touch the **Search** button.
4. Select a single test or multiple tests.
5. Touch the **Delete**, **Print**, **Edit**, or **Send** button to manage the saved tests.

Saving a test as a PDF

To save a test as a PDF

1. Touch the **Settings** tab. The ECG tab and the vertical ECG configuration tab appear.
2. Touch the **Auto report** tab.
3. Select *PDF to USB*, *PDF to remote file location*, *Internal and PDF to USB*, or *Internal and PDF to remote file location*, from the Printer drop-down menu.

Note If *Internal* or *Off* are selected, a PDF will not be created.

Managing the worklist

The worklist is a group of patients whose demographic data has been entered into the electrocardiograph memory to be recalled for a test later that day. The worklist holds up to 50 patients.

When you perform an auto ECG, you can fill in the patient's data from the worklist automatically by touching the Worklist tab.

You can add or delete patients any time. If the electrocardiograph is connected to a hospital information system (or an EMR), you can also download a worklist.

To manage the worklist

1. Touch the **Manage worklist** tab.
2. Touch the **Download** button, or touch the **Add** or **Delete** button to add or delete patients.

Settings

Viewing or changing the ECG settings

- The ECG settings control the content and format of your reports. These settings include a second Auto report format (Auto report) and a Rhythm format (Rhythm report), customizable patient data fields, and auto save options.

To view or change the settings

1. Touch the **Settings** tab. The ECG tab and the vertical ECG configuration tab appear.

Modify the settings as desired:

Note The following settings are saved as they are selected.

- Waveform centering on
- Baseline filter on
- Muscle filter on
- Save reminder on
- Default gain
- QTc method

Touch the  (Next) button.

Modify the settings as desired:

- Electrode labels
- Electrode configuration
- ECG interval
- Lead timing

Touch the  (Next) button.

Modify the settings as desired:

- Print preview on
- Test assignment on
- Test assignment reminder on
- ECG preview arrangement

Touch the **Rhythm report** tab.

Modify the settings as desired:

- Default speed
- Print options

Touch the  (Next) button.

Modify the settings as desired:

- Rhythm leads 1 - 12

Touch the **Auto report** tab.

Modify the settings as desired:

- Printer
 - Internal
 - PDF to USB
 - PDF to remote file location
 - Internal and PDF to USB
 - Internal and PDF to remote file location
 - Off
- Lead arrangement
- Average cycles
- Rhythm leads 1 - 3

Touch the  (Next) button.

Modify the settings as desired:

- First name
- Abnormal ECG
- Unconfirmed report
- Interpretation

- Middle Initial
- Height
- Weight
- Race

Select:

- Age or Birth date

Touch the  (Next) button.

Modify the settings as desired:

- Extended measurements
- MEANS reason statements (optional purchase)
- Comments

Selecting the device information

To select the device information

1. Touch the **Settings** tab. The ECG tab appears.
2. Touch the **Device** tab. The Regional tab appears.

Modify the settings as desired:

- Date format
- Time format
- Height
 - cm
 - ft & in
 - in
- Weight
 - kg
 - lb
- Mains (AC) frequency
- Language

Touch the **Device configuration** tab.

Modify the settings as desired:

- Auto save
- Data management
- LCD brightness

- HR beep on
- Error beep on
- Caps lock on or off
- Calibrate

Touch the **About** tab to view:

- Device serial number, model, software version, memory, battery charge percentage, etc.

Touch the **Network** tab to view or modify network settings. See *To connect to a network-connected workstation via Ethernet cable* for further details.

If you need help, consult your network administrator.

Touch the **Ownership** tab.

Modify the settings as desired:

- Practice ID
- Device ID

- Contact information

Touch the **Updates** tab.

- Touch **Update** to install the latest device software.
- Touch **Restore** to restore factory defaults.
- Touch **Save configuration** to save the device configuration to USB.
- Touch **Copy configuration** to copy the configuration from USB.

Maintenance

Cleaning the equipment



WARNING Keep the electrocardiograph, reusable electrodes, and the patient cable clean. Patient contact with contaminated equipment can spread infection.



CAUTION Never let soap or water come into contact with the electrocardiograph's internal printer, connectors, or jacks.



CAUTION Never immerse the electrocardiograph or the patient cable in liquid. Never autoclave or steam clean the electrocardiograph or the patient cable. Never pour alcohol directly on the electrocardiograph or the patient cable, and never soak any components in alcohol. If any liquid enters the electrocardiograph, remove the electrocardiograph from service, and have it inspected by a qualified service person before using it again.

Note The patient cables should be cleaned after each use.

Clean on a routine basis according to your facility's protocols and standards or local regulations.

The following agents are compatible with the electrocardiograph:

- 70 percent isopropyl alcohol
- 10 percent chlorine bleach solution




CAUTION When cleaning the device, avoid using cloths or solutions that include quaternary ammonium compounds (ammonium chlorides) or glutaraldehyde-based disinfectants.




Note Disinfect according to your facility's protocols and standards or local regulations.

To clean the equipment (monthly or more often if needed)

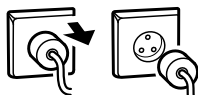1.



Disconnect the power plug from the AC outlet.

-
2.  Turn off the electrocardiograph. (Press and hold the power button for at least six seconds until the screen goes blank.)
-
3.  Dampen a cloth with any of the acceptable cleaning solutions, and wipe the exterior of the patient cable and electrocardiograph. Dry all components with a clean, soft cloth, or paper towel.
-
4.  Before you turn on the electrocardiograph again, wait at least 10 minutes for all traces of liquid to evaporate.
-

70 percent isopropyl alcohol

Wipe the electrocardiograph with a clean cloth slightly dampened with 70 percent isopropyl alcohol.

10 percent chlorine bleach solution

1. Wipe the electrocardiograph with a clean cloth slightly dampened with a 10 percent bleach and water solution. Follow the cleaning agent manufacturer's guidelines.
2. Rinse with a clean cloth slightly dampened with water that meets EP and USP quality standards.
3. Allow the electrocardiograph surface to dry for a minimum of 10 minutes before using the electrocardiograph.

Inspecting the equipment

Perform the following inspections daily.

- Check for cracked or broken patient cable, patient electrodes, power cord, communications cables, display, and enclosure.
- Check for bent or missing pins on all cables.
- Check all cable and cord connections; reseat if any connectors are loose.

Testing the electrocardiograph

Welch Allyn recommends verifying proper operation of the electrocardiograph once a year to ensure reliability. See *Verifying proper operation*.

Whenever the electrocardiograph is serviced or whenever problems are suspected, verify continued electrical safety of the device using IEC 60601-1 or ANSI/AAMI ES1 methods and limits.



WARNING Only qualified service personnel should perform leakage current tests.

Test for the following:

- Patient leakage current

- Chassis leakage current
- Earth leakage current
- Dielectric strength (AC and patient circuits)

Replacing the battery


Replace the battery under these circumstances:

- It loses its charge quickly.
- You have charged it, and the electrocardiograph still does not turn on when unplugged.

The first time that you press the power button after installing a new battery, the electrocardiograph goes through some diagnostic tests that cause it to take longer than usual to power up.

Discard the old battery appropriately. Contact your local authorities concerning recycling.

To replace the battery

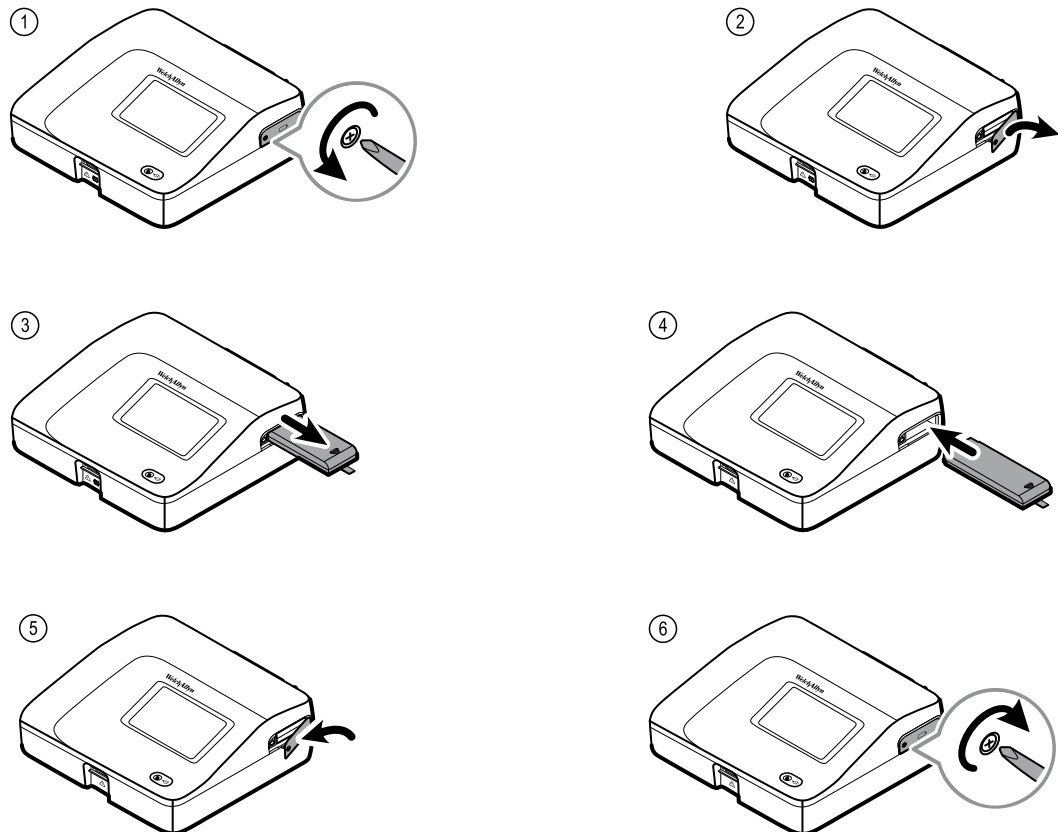


Replacing the AC fuses

You might need to replace one or both of the AC fuses if the AC power indicator does not light up when the electrocardiograph is connected to AC power.



WARNING Failure to unplug could result in electrocution.

To replace the AC fuses

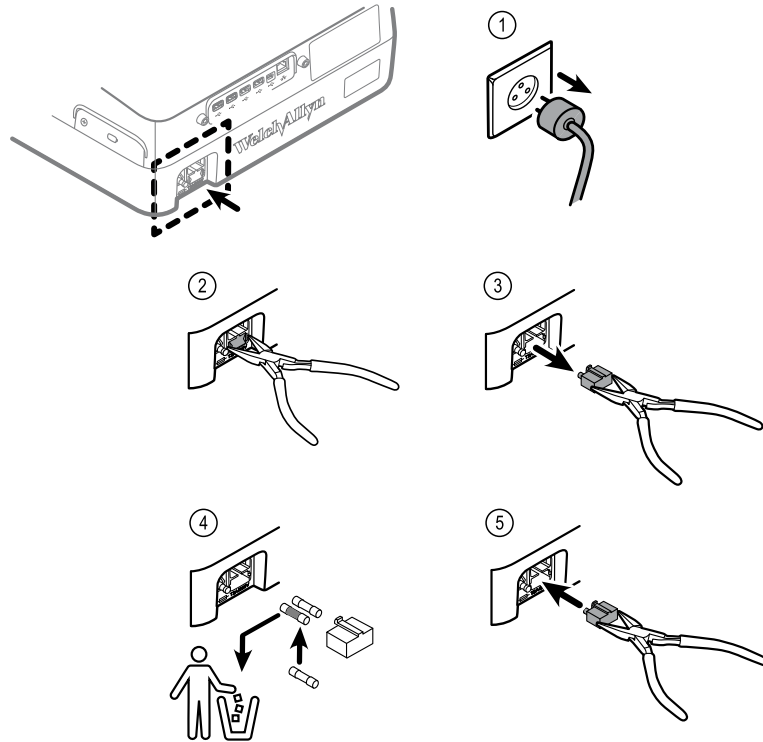

If either fuse is dark or has a broken wire, replace the fuse. Line up the fuse case with the opening; it goes in only one way.

Storing the equipment

When storing the electrocardiograph, cords, and accessories, observe the environmental storage conditions that are identified in the product specifications.

Discarding the equipment

Discard the electrocardiograph, cords, and accessories according to local laws.



Do not dispose of this product as unsorted municipal waste. Prepare this product for reuse or separate collection as specified by Directive 2002/96/EC of the European parliament and the Council of the European Union on Waste Electronic and Electrical Equipment (WEEE). If this product is contaminated, this directive does not apply. For more specific disposal information, see www.welchallyn.com/weee, or contact Welch Allyn Customer Service at +44 207 365 6780.

Troubleshooting

Lead-quality problems

“Artifact” message on the screen

Artifact is signal distortion that makes it difficult to accurately discern the waveform morphology.

Causes


- The patient was moving.
- The patient was shivering.
- There is electrical interference.

Actions

See actions for wandering baseline, muscle tremor, and AC interference.

Wandering baseline

Wandering baseline is an upward and downward fluctuation of the waveforms.

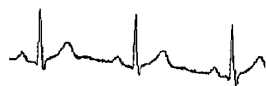

Causes

- Electrodes are dirty, corroded, loose, or positioned on bony areas.
- The electrode gel is insufficient or dried.
- The patient has oily skin or used body lotions.
- Rising and falling of chest during rapid or apprehensive breathing.

Actions

- Clean the patient's skin with alcohol or acetone.
- Reposition or replace the electrodes.
- Verify that the patient is comfortable, warm, and relaxed.
- If wandering baseline persists, turn the baseline filter on.

Muscle tremor



Causes

- The patient is uncomfortable, tense, nervous.
- The patient is cold and shivering.
- The exam bed is too narrow or short to comfortably support arms and legs.
- The arm or leg electrode straps are too tight.

Actions

- Verify that the patient is comfortable, warm, and relaxed.
- Check all electrode contacts.
- If interference persists, turn the muscle-tremor filter on. If interference still persists, the problem is probably electrical in nature. See the suggestions for reducing AC interference (in a related troubleshooting tip).

AC interference

AC interference superimposes even-peaked, regular voltage on the waveforms.



Causes

- The patient or technician was touching an electrode during recording.
- The patient was touching a metal part of an exam table or bed.
- A lead wire, patient cable, or power cord are broken.
- Electrical devices in the immediate area, or lighting, or wiring concealed in walls or floors are interfering.
- An electrical outlet is improperly grounded.
- The AC filter is turned off or set incorrectly.

Actions

- Verify that the patient is not touching any metal.
- Verify that the AC power cable is not touching the patient cable.
- Verify that the proper AC filter is selected.
- If interference persists, unplug the electrocardiograph from AC power and run it on the battery. If this solves the problem, you'll know that the noise was introduced through the power line.
- If interference still persists, the noise may be caused by other equipment in the room or by poorly grounded power lines. Try moving to another room.

Lead alert or square wave



A dot might be flashing on the lead-status screen. Or one or more leads might appear as a square wave.

Causes

- Electrode contact might be poor.
- A lead might be loose.
- A lead might be defective.

Actions

- Replace the electrode.
- Verify that the patient's skin has been properly prepared.
- Verify that electrodes have been properly stored and handled.
- Replace the patient cable.

System problems

The electrocardiograph won't turn on when it is plugged in

Causes

- The AC power connection is faulty.
- An AC fuse is blown.
- There is no AC power.

Actions

- Check the AC power source.
- Check the AC fuses.

The electrocardiograph won't turn on when it is unplugged

Causes

- The battery is disconnected or incorrectly connected.
- The battery is low, not charging, depleted, or bad.

Actions

- Check the battery connections.
- Recharge the battery.
- Replace the battery.

The electrocardiograph shuts down during printing

Causes

- The battery is low or bad.

Actions

- Recharge the battery.
- Replace the battery.

The electrocardiograph prints fewer than 10 reports on a full battery charge

Causes

- The battery is degraded.

Actions

- Replace the battery.

The electrocardiograph does not respond when you press buttons or touch the screen

Causes

- The electrocardiograph is frozen.

Actions

- Reset the electrocardiograph by pressing and holding the power button for at least six seconds until the screen goes blank. Press the power button again. The electrocardiograph goes through some diagnostic tests that cause it to take longer than usual to power up.
- Touch the **Settings** tab. Touch the **Advanced** tab. Touch the **Power down** button. The electrocardiograph goes through some diagnostic tests that cause it to take longer than usual to power up.

Note For more troubleshooting guidelines, see the service manual.



CAUTION The service manual is for use only by qualified service personnel who understand technical English.

Limited warranty

Welch Allyn warrants the product to be free of defects in material and workmanship and to perform in accordance with manufacturer's specifications for the period of three years from the date of purchase from Welch Allyn or its authorized distributors or agents.

The warranty period shall start on the date of purchase. The date of purchase is: 1) the invoiced ship date if the device was purchased directly from Welch Allyn, 2) the date specified during product registration, 3) the date of purchase of the product from a Welch Allyn authorized distributor as documented from a receipt from said distributor.

This warranty does not cover damage caused by: 1) handling during shipping, 2) use or maintenance contrary to labeled instructions, 3) alteration or repair by anyone not authorized by Welch Allyn, and 4) accidents.

The product warranty is also subject to the following terms and limitations: Accessories are not covered by the warranty. Refer to the directions for use provided with individual accessories for warranty information.

Shipping cost to return a device to a Welch Allyn Service center is not included.

A service notification number must be obtained from Welch Allyn prior to returning any products or accessories to Welch Allyn's designated service centers for repair. To obtain a service notification number, contact Welch Allyn Technical Support.

THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. WELCH ALLYN'S OBLIGATION UNDER THIS WARRANTY IS LIMITED TO REPAIR OR REPLACEMENT OF PRODUCTS CONTAINING A DEFECT. WELCH ALLYN IS NOT RESPONSIBLE FOR ANY INDIRECT OR CONSEQUENTIAL DAMAGES RESULTING FROM A PRODUCT DEFECT COVERED BY THE WARRANTY.

Service policy

All repairs on products under warranty must be performed by Welch Allyn or by a service provider authorized by Welch Allyn. Unauthorized repairs will void the warranty. In addition, whether or not covered under warranty, any product repair should be performed exclusively by Welch Allyn or by a service provider that has been authorized by Welch Allyn.

If the product fails to function properly—or if you need assistance, service, or spare parts—contact the nearest Welch Allyn Technical Support Center.

Before contacting Welch Allyn, try to duplicate the problem, and check all accessories to ensure that they are not causing the problem. When calling, please be prepared to provide:

- Product name, model number, and serial number of your product.
- Complete description of the problem.
- Complete name, address and phone number of your facility.
- For out-of-warranty repairs or spare parts orders, a purchase order (or credit card) number.
- For parts orders, the required spare or replacement part numbers.

If your product requires warranty, extended warranty, or non-warranty repair service, please call first the nearest Welch Allyn Technical Support Center. A representative will assist you troubleshooting the problem and will make every effort to solve it over the phone, avoiding potential unnecessary return of your product.

In case a return cannot be avoided, the representative will record all necessary information and will provide a Return Material Authorization (RMA) number, as well as the appropriate return address. An RMA number must be obtained prior to any return.

If you have to return your product for service, follow these recommended packing instructions:

- Remove all hoses, cables, sensors, power cords, and other accessories (as appropriate) before packing, unless you suspect they are associated with the problem.
- Wherever possible use the original shipping carton and packing materials.
- Include a packing list and the Welch Allyn Return Material Authorization (RMA) number.

It is recommended that all returned goods be insured. Claims for loss or damage to the product must be initiated by the sender.

EMC guidance and manufacturer's declarations

EMC compliance

Special precautions concerning electromagnetic compatibility (EMC) must be taken for all medical electrical equipment. This device complies with IEC EN 60601-1-2:2007.

- All medical electrical equipment must be installed and put into service in accordance with the EMC information provided in this document and the *Welch Allyn CP 150 12-lead resting electrocardiograph Directions for Use*.
- Portable and mobile RF communications equipment can affect the behavior of medical electrical equipment.

The electrocardiograph complies with all applicable and required standards for electromagnetic interference.

- It does not normally affect nearby equipment and devices.
- It is not normally affected by nearby equipment and devices.
- It is not safe to operate the electrocardiograph in the presence of high-frequency surgical equipment.
- However, it is good practice to avoid using the electrocardiograph in extremely close proximity to other equipment.

Emissions and immunity information

For information about electromagnetic compatibility (EMC), see the Welch Allyn website.

http://www.welchallyn.com/apps/products/product_category.jsp?catcode=CARDIO

Specifications

Item	Specification
Dimensions, including rubber feet (length x width x height)	380.9 mm x 358.1 mm x 136.2 mm (15 x 14.1 x 5.4 in.)
Weight , including battery	5.2 kg (11.5 lb)
Keyboard type (power button)	Polyester overlay
Display	
Type	TFT, 18 cm (7 in.) color touch screen
Resolution	WVGA, 800 x480
Thermal paper	
Z-fold	8.25 x 11 inches (21 x 28 cm), 200 sheets
Thermal printer (internal)	Computer-controlled dot array, 8 dots/mm
Thermal-chart-paper speeds	10, 25, 50 mm/s
Gain settings	
Auto ECGs	2.5, 5, 10, 20 mm/mV, Auto
Rhythm ECGs	2.5, 5, 10, 20 mm/mV
Lead configurations	Standard, Cabrera
Report formats, internal printer, Auto report*	3x4-2.5s @ 25 mm/s 3x4-2.5s @ 50 mm/s 3x4+1R-2.5s @ 25 mm/s 3x4+3R-2.5s @ 25 mm/s 3x4-5.0s @ 25 mm/s 3x4-5.0s @ 50 mm/s

Item	Specification
	6x2-5.0s @ 25 mm/s 6x2-5.0s @ 50 mm/s 12x1-10.0s @ 25 mm/s
Report formats, internal printer, Average	3x4+3R @ 25 mm/s 3x4+3R @ 50 mm/s 6x2+1R @ 25 mm/s 6x2+1R @ 50 mm/s No print
ECG storage (in test directory)	At least 100 ECG tests
Patient storage	Up to 50 patients
Frequency range	0.3 to 150 Hz
Digital sampling rate	> 1,000 samples/second/channel
Pacemaker detection	ANSI/AAMI EC11
Power requirement	Universal AC power supply 110-240 V~, 50/60 Hz ~, 1.5 A maximum
AC fuses	Time-lag type, 2.0-amp 250-V rating, Littelfuse 0218002P or equivalent
Rechargeable battery	9 cells
Rating	10.8 V 6.75Ah (73Wh)
Composition	Lithium-ion
Recharge time to 100 percent capacity	4 hrs
Full charge capacity	25 ECG tests @ 20 minutes per test
	8 hrs of continuous operation or 250 continuous ECGs
Filters	
High-performance baseline	0.5 Hz
Muscle tremor	35 Hz
AC interference	50 Hz or 60 Hz
Safety, EMC, and regulatory compliance	ANSI/AAMI EC11** CAN/CSA C22.2 No. 601.1 CAN/CSA C22.2 No. 601.1.2
	IEC/EN 60601-1 IEC/EN 60601-1-2 IEC/EN 60601-1-4

Item	Specification
	CAN/CSA C22.2 No. 601.1.4 IEC/EN 60601-1-6 CAN/CSA C22.2 No. 601.2.25 IEC/EN 60601-2-25 *** IEC/EN 60601-2-51* (3x4 report format)
Standard connectivity	1 USB client 4 USB hosts
Connectivity with electronic medical records	Through the Welch Allyn CardioPerfect workstation software
Electrodes	Rigorously tested for conductivity, adhesion, and hypoallergenic qualities, and exceed all AAMI standards
Power cable	Meets or exceeds Type SJT.
Environmental operating conditions	
Temperature	+10° C to +40° C (+50° F to +104° F)
Relative humidity	15 - 95% noncondensing (30 - 70% for printing)
Atmospheric air-pressure limits	700 - 1060 hPa
Environmental storage conditions	
Temperature	-20° C to +50° C (-4° F to +122° F)
Relative humidity	15 - 95% noncondensing
Atmospheric air-pressure limits	700 - 1060 hPa
Protection against electric shock	Class I, internally powered Type CF
Mode of operation	Continuous

* If you print at a high gain setting, the waveform or calibration marks might be clipped. This clipping does not comply with clause 51.103.1 of the IEC/EN 60601-2-51 standard. Use a lower gain setting to observe the full waveform.

** Per AAMI EC11:1991/®)2007 Diagnostic Electrocardiographic Devices, Section 3.1.2.1 Disclosure of cautionary information/ performance characteristics paragraph c) Accuracy of input signal reproduction, the manufacturer shall disclose the methods used to establish overall system error and frequency response. Welch Allyn has used methods A & D, as prescribed in section 3.2.7.2 and 4.2.7.2 of this same standard, to verify overall system error and frequency response. Because of the sampling characteristics and the asynchronism between sample rate and signal rate, digital ECG systems such as the CP 150 may produce a noticeable modulating effect from one cycle to the next, particularly in pediatric recordings. This phenomenon is not physiologic.

*** Disposable electrodes from Welch Allyn shall be used during patient defibrillation.

Specifications are subject to change without notice.

WelchAllyn®

Advancing Frontline Care™



720157

Material No.

720157